

Invia® Wound Therapy

Wound Dressing Sets



“What I want...
is an effective
wound treatment
for my patients.”



Invia® Wound Dressing Sets offer the most options which provide flexibility in clinical applications based upon the wound characteristics

Invia® dressing sets includes:

- **Kerlix™ AMD™ Antimicrobial gauze:** as a primary layer AMD works as a contact kill maintaining log counts; as a secondary dressing limits cross-contamination by reducing bacterial growth in pooling layer
- **3M™ Tegaderm™ Non-adherent contact layer:** placed directly in the wound bed to lessen trauma to new tissue and minimize patient discomfort
- **Wound drain:** external or internal offering flexibility to fit the wound characteristics used to retain negative pressure within the sealed wound environment as needed
- **3M™ Tegaderm™ Transparent dressing:** waterproof film to protect the integrity of the wound
- **3M™ Cavilon™ barrier film wipes:** improves adhesion of transparent dressing and provides protection for the surrounding skin from adhesive trauma
- **Hy•Tape® Surgical adhesive tape:** latex-free, waterproof tape with a zinc oxide base designed to be soothing to delicate skin; may be used to border dressing as needed or to affix drain to body
- **Sterile saline:** irrigate and cleanse the wound prior to applying dressing and saturate the gauze
- **Coloplast® Ostomy strip paste:** used to fill crevices or uneven surfaces to improve seal in difficult to dress areas

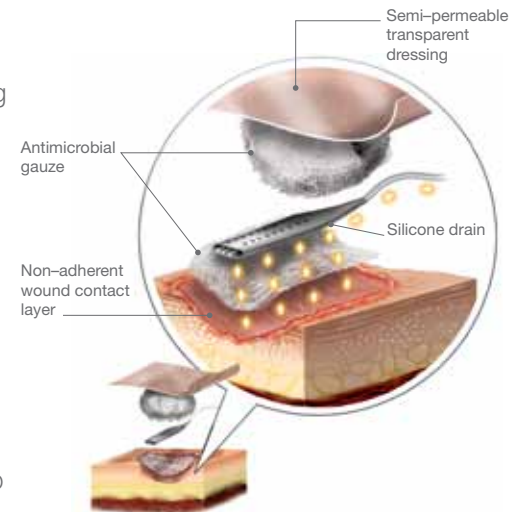


Image	Invia Drains	Features and Benefits	Depth			Special Conditions		Drainage		
			Shallow	Medium	Deep	Tunnel	Undermining	Minimal	Moderate	Heavy
	External Suction Interface (ESI)	<ul style="list-style-type: none"> • Multipurpose applications, ideal for difficult to seal areas in non weight bearing wounds • Designed to adhere to transparent dressing for a complete seal 	✓	✓	✓			✓	✓	✓
	10mm Flat	<ul style="list-style-type: none"> • Multipurpose applications, ideal for non weight bearing wounds without particulate • Soft and flexible allowing for easy placement within wound bed • Radiopaque silicone allows for X-ray detection 	✓	✓	✓			✓	✓	✓
	15Fr Round Channel	<ul style="list-style-type: none"> • Multipurpose applications, ideal for tunneling and undermining • Designed to efficiently channel drainage • Channels help prevent clotting or occlusions • Minimizes tissue in-growth • Radiopaque silicone allows for X-ray detection 		✓	✓	✓	✓	✓	✓	✓
	10Fr Round Perforated	<ul style="list-style-type: none"> • Ideal for undermining or laying linear within wound bed without particulate in weight bearing areas • Flexible to coil within the wound bed • Perforations allow effective removal of drainage 	✓	✓			✓	✓	✓	✓
	19Fr Round Perforated	<ul style="list-style-type: none"> • Ideal for large deep wounds without particulate in weight bearing areas • Largest size drain available to accommodate heavy exudating wounds • Perforations allow effective removal of drainage 	✓	✓	✓			✓	✓	✓

Note: Please consider the patient's activities of daily living and anatomical location of the wound when selecting a drain.

Rental and Ordering Information

Joerns Healthcare has customer service representatives available 24 hours a day, 7 days a week, to assist you with your needs, simply call us at **1.800.826.0270** and select option #1. Please send all faxed orders to 800.232.9796.

Distributed by:



Joerns Healthcare
5001 Joerns Drive
Stevens Point, WI 54481
Phone: 800-826-0270
Fax: 800-232-9796
www.joerns.com

Manufactured by:



Medela, Inc.
1101 Corporate Drive, McHenry, IL 60050
Phone: 877-MY INVIA (877-694-6842)
Fax: 815-363-2487
Email: myinvia@medela.com
www.myinvia.com