



Joerns Healthcare

# Arise® 1000 and Arise 1000EX

## Equipment Information Sheet

### Directions For Caregivers

The Arise® 1000 and Arise 1000EX replaces the current mattress on your hospital bed frame.

The Arise 1000 and Arise 1000EX is designed to meet pressure relief needs of patients up to 1000 lbs in weight with pressure ulcers and those at risk of skin breakdown. Low air loss therapy has been demonstrated to be a valuable aid in the treatment of pressure ulcers.

### Arise 1000 Control Unit

#### Power Switch

Used to turn the power on and off.

#### Lockout Feature

Prevents unauthorized changes to patient settings. Five (5) minutes after the last mode adjustment, the *Power* light begins to flash indicating *Lockout* is enabled. You must release the lockout in order to change the therapy mode. Unlock by pressing both ↑ and ↓ *Comfort Adjust* arrows simultaneously.

#### Standby

When light is on, press the *Power* button and reset preferred mode of therapy and comfort level.

#### Comfort Adjust Buttons

The *Comfort Adjust* is located on the right side of the control panel. The system can be customized to meet individual patient needs within a therapeutic window. These buttons allow you to control the amount of air, or firmness, to the mattress. Use ↑ and ↓ *Comfort Adjust* arrows to increase or decrease pressure.

#### Alarm Reset

If power fails, the control unit will alarm audibly and the indicator light will illuminate. To cancel the alarm press *Alarm reset*.

#### CPR

The hose connection at the control unit is marked *CPR*. Disconnect the hose from the control unit. Deflation times will vary based on patient weight and profile. To resume therapy, reattach the hose to the control unit.

## MODES

**Autofirm:** To quickly inflate the mattress to maximum pressure, use *Autofirm* mode. This mode may be desirable to assist with patient transfers and other nursing procedures. Therapy system will automatically return to the *Static/Flotation* mode after 12 minutes.

**Therapy:** In this mode, the mattress provides full body flotation and remains static, allowing for patient immersion and flotation.

**Pulsate:** *Pulsate* will slightly decrease the pressure in all therapy cells every 30 seconds, which some patients find comfortable.

**Back Raise Sensor:** If the head of the mattress is raised above 25 degrees, the LED will turn on and the system will increase the pressure in the mattress to compensate for increased pressure in the torso section.

## TROUBLESHOOTING

### Therapy surface is not inflated

- Ensure that the control unit is plugged into an AC outlet.
- Ensure hose from therapy mattress to the control unit is securely connected.
- Check the *Standby* light. If it is not on, press *Power*.

### Unable to change the therapy mode or adjust comfort controls

- Make sure that the *Lockout* function is disabled. To disable, press the ↑ and ↓ *Comfort Adjust* arrows simultaneously.

## POWER FAILURE

**Power Interruption:** In a power interruption, the lower support cell of the Arise 1000 will hold air for up to 12 hours. For added safety, the lower three-inch foam support cell of the Arise 1000EX provides additional therapeutic support and remains supportive in the event of a power failure or for patient transport. The control unit will flash a red *Alarm* light and sound an audible signal while keeping current settings in memory for up to 20 minutes.

**Power Loss:** In a power loss lasting over 20 minutes, the *Standby* light will illuminate after power restoration. Press the *Power* button and reset the preferred mode and comfort level.

**If you have any questions, please call  
Customer Care at 800.826.0270 US • (+31) 30.6363.700 EU**

© 2011 Joerns Healthcare • 6220131 RevC • 11-0032