



### Directions For Caregivers

The Cair 1000/Cair 1000 EX replaces the current mattress on your bed frame.

The Cair 1000/Cair 1000 EX is designed to meet pressure redistribution needs of patients up to 1000 lbs. with pressure ulcers and those at risk of skin breakdown.

### Cair 1000 Control Unit

#### Power Switch

When the control unit is plugged in the *Standby* light will illuminate. Press the *Power* button to turn on the system.

#### Lockout Feature

The *Lockout* feature prevents unauthorized changes to patient settings. Five minutes after the last mode adjustment, the *Power* light begins to flash indicating *Lockout* is enabled. You must release the *Lockout* in order to change the therapy mode. Unlock by pressing both ↑ and ↓ *Therapy Adjust* arrows simultaneously.

#### Therapy Adjust Buttons

The *Therapy Adjust* buttons allow you to control the amount of air, or firmness, to the mattress. Use the ↑ and ↓ *Therapy Adjust* arrows to adjust the pressure.

#### CPR

The hose connection at the control unit is marked *CPR*. Disconnect the hose from the control unit. Deflation times will vary based on patient weight and profile. To resume therapy, reattach the hose to the control unit.

### MODES

#### Autofirm

To quickly inflate the mattress to maximum firmness, use *Autofirm* mode. This mode may be desirable to assist with patient transfers and other nursing procedures. The system will automatically return to the *Therapy* mode after approximately 12 minutes.

## Therapy

The *Therapy* mode provides full body flotation via 20 static support cells.

## Pulsate

*Pulsate* will oscillate the air in all therapy cells every 30 seconds, from the current setting to the next lowest setting and back.

## Position Sensor

If the head of the mattress is raised above approximately 30 degrees, the LED will illuminate and the system will increase air support in the mattress to compensate for increased weight in the seat section.

# TROUBLESHOOTING

## Therapy surface is deflated

- Ensure that the control unit is plugged into an A/C outlet.
- Ensure hose from the mattress to the control unit is securely connected.
- Make sure that the *Standby* light is not illuminated. If the *Standby* light is illuminated press the *Power* button.

## Unable to change the therapy mode or adjust comfort controls

Make sure that the *Lockout* function is disabled

- To disable, press the ↑ and ↓ *Therapy Adjust* arrows simultaneously.

# POWER FAILURE

In the event of power failure, the support cell will hold air for up to 12 hours.

**Power Interruption:** The control unit will flash a red *Alarm* light and sound an audible signal while keeping current settings in memory for up to 20 minutes.

**Power Loss:** In a power loss lasting over 20 minutes, the *Standby* light will illuminate after power restoration. Press the power button and reset the preferred mode and comfort level.

**Alarm Reset:** The *Alarm Reset* light will continue to flash after power is restored. To disengage the notification, press the *Alarm Reset* button.

**If you have any questions, please call  
Customer Care at 800.966.6662**