


SCALE SYSTEM FUNCTIONS AND WEIGHING PROCEDURES

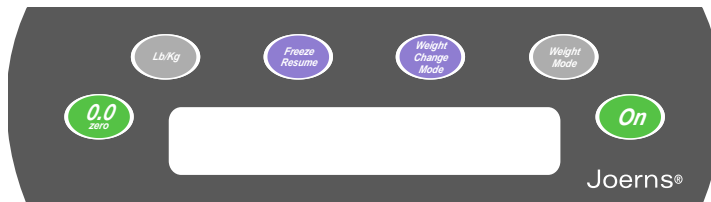

⚠ Caution: If present, 5th Wheel must be disengaged to obtain an accurate resident/patient weight.

0.0 Zero

With all bed linens in place and without resident/patient in bed, press **Zero** button. Displayed message will read "PLEASE WAIT - HANDS OFF". Ensure nothing touches weight-sensitive portion of integrated Scale System while performing zeroing function. In a few seconds, display will indicate "WEIGHT = 0.0 LB".

Lb/Kg

Weight data may be viewed in either **pounds** or **kilograms**. Pressing this button allows staff to switch between display modes quickly and easily. Both modes will display resident/patient's weight within 0.1 units of resolution (0.1 Lb. or 0.1 Kg.).

Freeze/Resume

Freeze/Resume mode button provides customized ability to adjust resident/patient's comfort by adding/ removing a pillow or blanket or ancillary equipment, without affecting resident/patient's displayed weight.

Press **Freeze/Resume** button. The display will read "FREEZE = (resident/patient's weight)." Make all necessary adjustments to bed or resident/patient. When done, press **Freeze/Resume** button again to resume normal weighing.

Weight Change Mode

This mode is used to monitor resident/patient's weight changes. **Note:** To enable this function initially, press

Weight Change Mode button with resident/patient in bed and then press **Zero** button to zero out weight. Display will read "WT CHG = (resident/patient weight)". Any variation in weight will now be viewed as a "+" or "-" figure relative to **zero**.

Weight Mode

If display shuts off before staff can view weight data, simply press the **On** button to refresh display and view resident/patient's weight. Display will indicate "WEIGHT = XXX.X LB"(or KG).

⚠ Caution: After initialization process is complete and Scale System is still in **Weight Change Mode**, **do not** press **Zero** button **again**, or all stored resident/patient's weight history data will be lost.

Pressing **Weight Mode** button will switch back to the **Weight Mode** allowing staff to view resident/patient's total weight again.

⚠ Caution: If the **Freeze/Resume** button is not pressed before adding/removing items from bed, the weight of those items will be reflected in the displayed weight, which may be hard to adjust back to the correct stored weight.


Note: A weight discrepancy described above can be corrected by using **Weight Mode** (for increases) and **Weight Change Mode** (for decreases) buttons to manually adjust displayed weight reading. You will need to know the weight of added/removed items to add/subtract from total weight.

If **Freeze/Resume** mode is left unattended for more than three minutes, display will read "LEFT IN FREEZE - REFER TO MANUAL." Press **Freeze/Resume** button to exit this mode.

On


Button refreshes display or initiates power for scale use.

STAFF CONTROL OPERATION

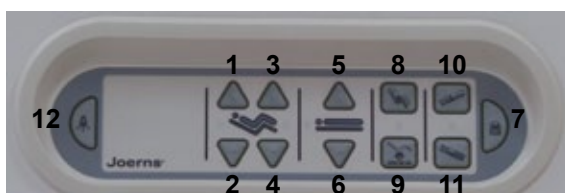

1. Push button to raise back
2. Push button to lower back
3. Push button to raise knee
4. Push button to lower knee
5. Push button to raise bed
6. Push button to lower bed
7. Push and hold to change lockout status in combination with another function button.
8. Push button for Comfort Chair Position
9. Push button to lower both back and knee to flat and place bed at low height
10. Push button for Trendelenburg
11. Push button for Reverse Trendelenburg
12. Push button for staff control backlighting

MOBILITY OPERATION

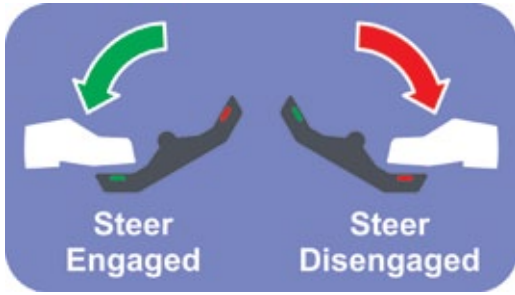

Fifth Wheel: Push pedal towards the green arrow to engage steering. Push pedal towards the red arrow to disengage steering.



Castors: Push pedal down to lock caster. Push pedal up to unlock. **Note:** Must lock/unlock each caster individually.

EXTENSION OPERATION



Length Extension: Slide the frame extender out by pulling the latch knob on the side of the bed frame. Next, slide the deck extender by pulling the latch knob underneath the deck and sliding the extender out.

Width Extension: Pull purple latch handle along the bottom edge of each section and pull.

ADDITIONAL FEATURES



CPR Release: Rotate red handle up and hold until deck is flat. Located at the foot end, patient left.



Angle Indicator: The angle indicator allows caregivers to read the TRT angle. Located at head of bed and both sides.



Side Rail Release: Pull purple handle towards you and rotate side rail down.