


# Hoyer®

## FULL BACK DISPOSABLE SLING

### USER GUIDE FOR THE FULL BACK DISPOSABLE SLING

The Full Back Disposable Sling is an easy fit, contoured sling designed to fit 85-90% of clients. It is designed for use as the traditional Quickfit Sling but offers greater comfort and negates the need for side suspenders. It has integrated, boned head support and leg padding as standard.

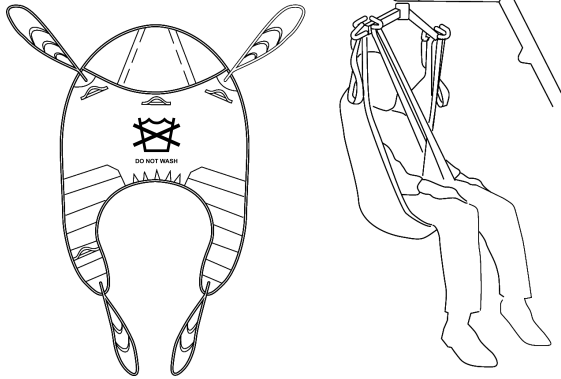

**NOTE: DO NOT WASH DISPOSABLE SLINGS**



**HOYER RECOMMENDS THE USE OF GENUINE HOYER PARTS.**

*Hoyer slings and lifters are not designed to be interchangeable with other manufacturer's products. Using other manufacturer's products on Hoyer products is potentially unsafe and could result in serious injury to patient and/or caregiver.*

*For the safety of the patient and carer, before using a sling a full risk assessment must be conducted to ensure that the correct sling choice, method of positioning in the sling and procedure for transfer has been determined for the patient.*

*Check sling and stitching before each use. Using bleached, torn, cut, frayed or broken slings is unsafe and could result in serious injury or death to the patient.*

*Destroy and discard worn slings. Do not alter slings.*


*Never leave a patient unattended.*




ADDITIONAL SLINGS ARE AVAILABLE TO SUIT INDIVIDUAL NEEDS.  
YOU ARE ADVISED THAT YOU SHOULD ALWAYS SEEK THE ADVICE OF AN  
AUTHORISED DEALER BEFORE PURCHASING.

WE WILL BE PLEASED TO ADVISE YOU OF YOUR NEAREST AUTHORISED DEALER.


## HOW TO FIT FROM SEATED POSITION

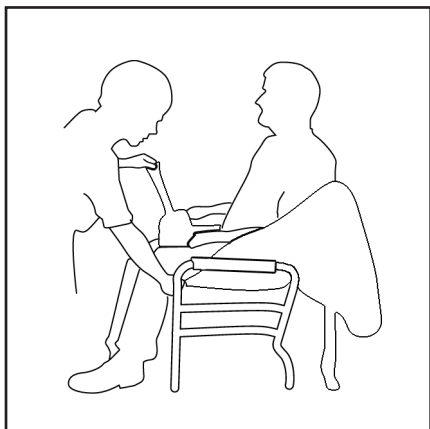

Raise the client's leg and feed the padded leg piece under and up between the legs. Ensure that the sling is not twisted or creased under the thigh. Repeat this procedure for the other leg.

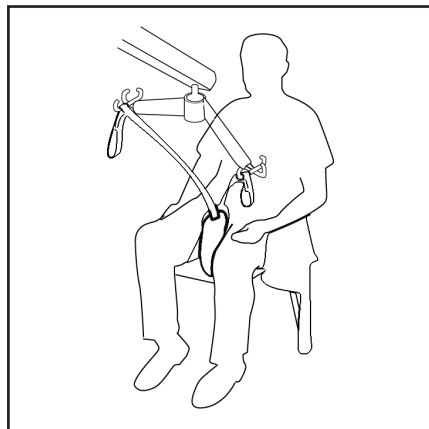

Feed the right leg support through the strap on the left leg support. Cross over leg straps, pass one leg strap through the other and attach to hoist on front hooks.

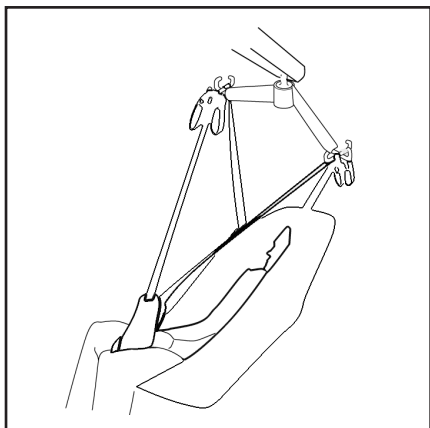


Raise lift to convenient height and attach the shoulder straps as shown. You may then raise the client to the required height. When seating the client the hand grip on the back of the sling may be used to ensure a good seated posture.

### **GOLDEN TIPS**


- 1. THE CLIENT'S POSTURE CAN BE ALTERED BY ADJUSTING BOTH SHOULDER STRAPS.**
- 2. IF IN DOUBT, ATTACH TO COLOURED LOOPS.**
- 3. KEEP SHOULDER STRAPS AT EQUAL LENGTH.**

## HOW TO FIT SLING FROM LYING POSITION

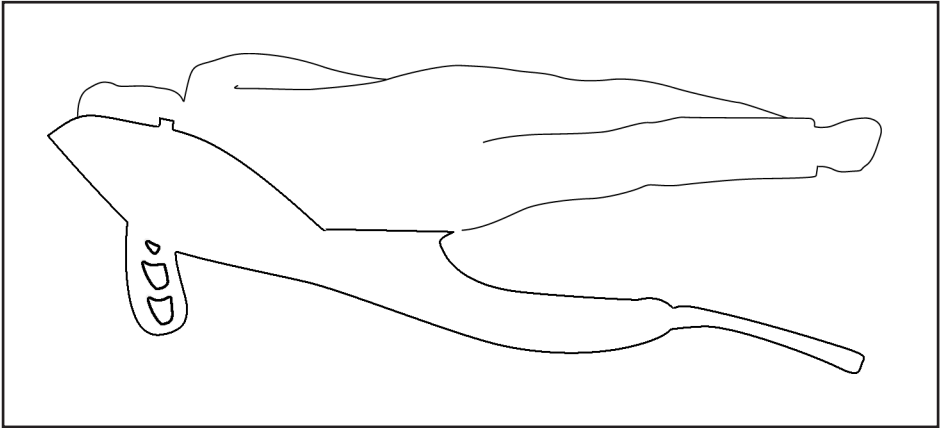

Draw sheet roll the client onto the sling, ensuring that the top of the commode aperture is at the base of the spine.

Bring the leg support straps up and between the client's legs and proceed as from the seated position, attaching to the shortest possible loops. N.B. If you wish to place the client onto a high bed it may be necessary to lower the patient onto an intermediate surface and adjust the strap length.

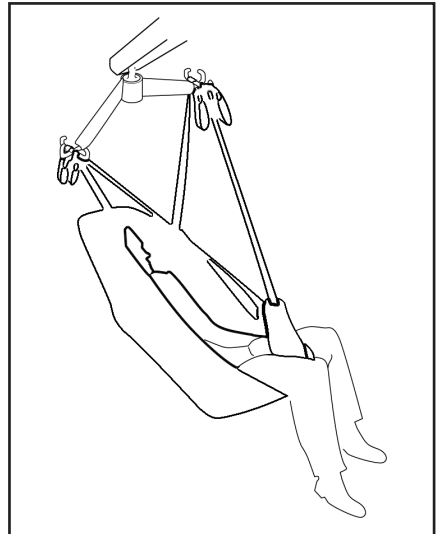


## FULL BACK DISPOSABLE SLING TECHNICAL SPECIFICATIONS

STANDARD	BS EN ISO 10535
SAFE WORKING LOAD	500lbs / 227kg

### SIZING

Each sling is marked with the following colour coded symbols, one of which will be crossed. Your sling is the uncrossed size, i.e. in this example Small (Red).

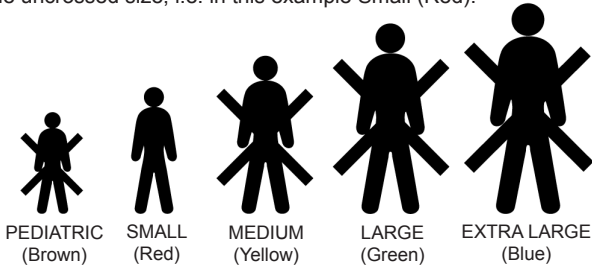

### WASHING INSTRUCTIONS

#### DO NOT WASH

Disposable slings are patient specific and are designed not to be washed. Washing will compromise the integrity of the sling and therefore make it unsafe for use.



<b>ORDER CODE</b>	PAEDIATRIC (BROWN)	NA25620 (Polyester)
	SMALL (RED)	NA25621 (Polyester)
	MEDIUM (YELLOW)	NA25622 (Polyester)
	LARGE (GREEN)	NA25623 (Polyester)
	EXTRA LARGE (BLUE)	NA25624 (Polyester)

DO NOT EXCEED THE RATED CAPACITY OF THE SLING OR THE LIFT.



Joerns Healthcare  
 2100 Design Road, Suite 100  
 Arlington, TX 76014  
 Tel: 800-826-0270  
 Fax: 800-457-8827  
[www.joerns.com](http://www.joerns.com)