User Instruction Manual Hoyer® Long Seat Sling

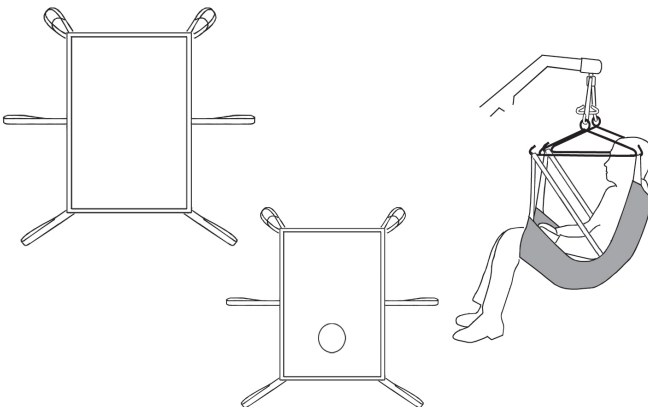
To avoid injury, read user manual prior to use.

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Introduction

The Hoyer® Long Seat sling is a hammock style sling designed for amputees but also suitable for other patients. This sling is traditionally fitted from a recumbent position and the use of side suspenders will facilitate a more comfortable transfer. Patients should be carefully assessed for suitability by a qualified person.

Statement of Intended Use

A sling is an item of moving and handling equipment that is used with a mechanical lift in order to facilitate the transfer of a patient. It comprises a specially designed and constructed piece of fabric that is placed under and/or around a patient before being attached to the spreader bar/cradle of a lift to raise, transfer and lower the patient. When selected and used correctly, a sling and lift combination will achieve a safer transfer and reduce the risks associated with manual handling.

It is the responsibility of a competent person to conduct a thorough risk assessment prior to using any sling, to ensure that the sling choice, method of positioning in the sling and procedure for transfer has been correctly determined for the patient. For further guidance, please contact your authorized Hoyer service provider or Joerns Healthcare directly.

FOR USE WITH PASSIVE LIFTS ONLY.

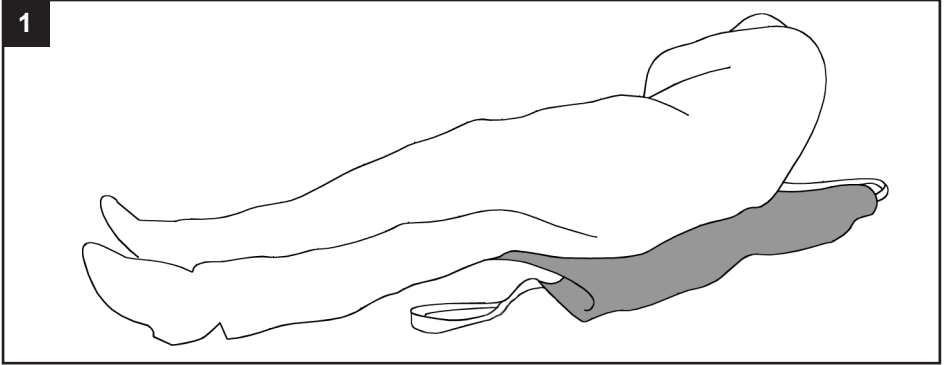
Expected Service Life

The expected service life of a Hoyer sling will vary dependent on use and following the care and washing instructions provided in the user guide. Factors such as wash temperature, detergents, frequency of use and patient weight will impact on the lifetime of your sling. Joerns Healthcare recommend that slings are checked each and every time prior to use to ensure the safety of the patient. Bleached, torn, cut, frayed or broken slings are unsafe and must be discarded and replaced.


⚠ WARNING

- **HOYER RECOMMENDS THE USE OF GENUINE HOYER SLINGS. Hoyer slings and lifters are not designed to be interchangeable with other manufacturer's products. Using other manufacturer's products on Hoyer products is potentially unsafe and could result in serious injury to patient and/or caregiver.**
- **For the safety of the patient and carer; before using a sling a full risk assessment must be conducted to ensure that the correct sling choice, method of positioning in the sling and procedure for transfer has been determined for the patient.**
- **CHECK sling and stitching before each use. Using bleached, torn, cut, frayed or broken slings is unsafe and could result in serious injury or death to the patient.**
- **DO NOT alter slings. Destroy and discard worn slings.**
- **NEVER leave a patient unattended.**
- **DO NOT exceed the rated capacity of the sling or lift.**
- **DO NOT attempt to re-position a patient by pulling on the sling loops.**

How to Fit A Long Seat Sling From A Recumbent Position

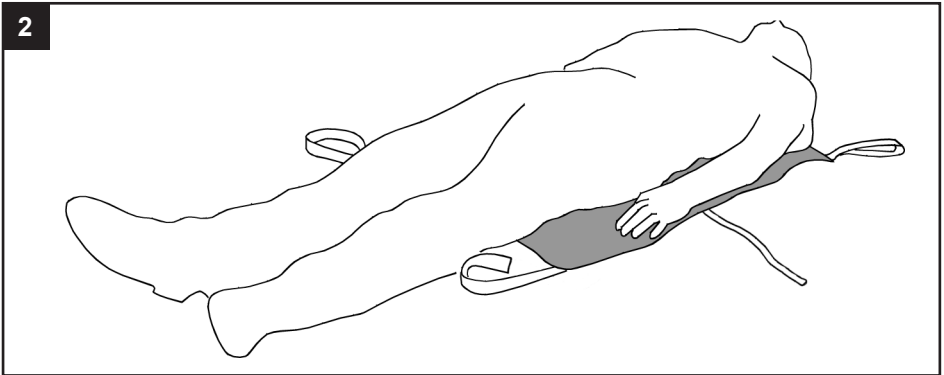
From the recumbent position, roll the patient onto the sling using the draw sheet roll. Ensure the bottom of the sling is just behind the knees.

Ensure the sling is laid flat underneath the patient. The top of the sling should be at shoulder blade height and level across the top and bottom edges.


3

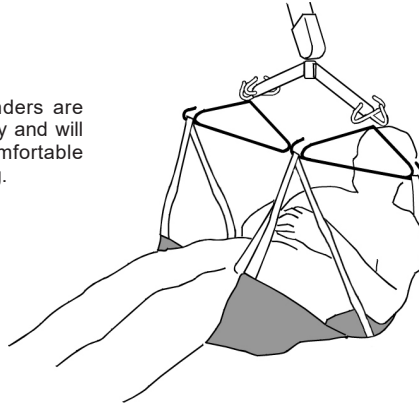NOTE: Side suspenders are an optional accessory and will facilitate a more comfortable transfer with this sling.

Move the lift into position above the patient. Attach the side suspenders. Keep the side suspenders away from the patients face. Attach the shoulder strap and leg loops to the respective front and rear hooks on the side suspenders. Finally, attach the hip straps to the respective front hooks on the side suspenders.

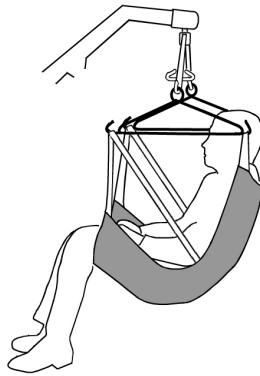4

Raise the lift to the required height. **NOTE:** It may be necessary to support the head while the patient is suspended.

Joerns Healthcare recommend slings are checked regularly and before use for fraying or damage.

Technical Specifications

STANDARD	BS EN ISO 10535
----------	-----------------

Sizing & Safe Working Load

SIZE	Long Seat (Polyester)	Long Seat (Net)	Long Seat Commode (Poly)	Long Seat Commode (Net)
EXTRA SMALL (BROWN)	500lbs / 227kg	500lbs / 227kg	500lbs / 227kg	500lbs / 227kg
SMALL (RED)	500lbs / 227kg	500lbs / 227kg	500lbs / 227kg	500lbs / 227kg
MEDIUM (YELLOW)	500lbs / 227kg	500lbs / 227kg	500lbs / 227kg	500lbs / 227kg
LARGE (GREEN)	500lbs / 227kg	500lbs / 227kg	500lbs / 227kg	500lbs / 227kg
EXTRA LARGE (BLUE)	500lbs / 227kg	500lbs / 227kg	500lbs / 227kg	500lbs / 227kg

Washing Instructions

DO NOT EXCEED 185°F water temperature. Machine wash in warm water.

DO NOT wash with bleach. Bleach will damage the sling's material and make it unsafe for use.

Cool tumble dry, air dry or dry at very low temperature.

DO NOT dry clean.

▲ WARNING

Slings can suffer damage during washing and drying and should be checked carefully before each use.

PLEASE NOTE: Additional slings are available to meet individual patient needs. We advise that you always seek the advice of a trained clinician, authorized Hoyer distributor or Joerns Healthcare prior to purchase or use.

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