


Hoyer®

Standing Disposable Sling

USER INSTRUCTION MANUAL

The Standing Disposable sling is an easy fit sling suitable for standing and supporting. It is not a general purpose sling and clients should be assessed as to their suitability. It will not suit double amputees or lifting from a horizontal position.

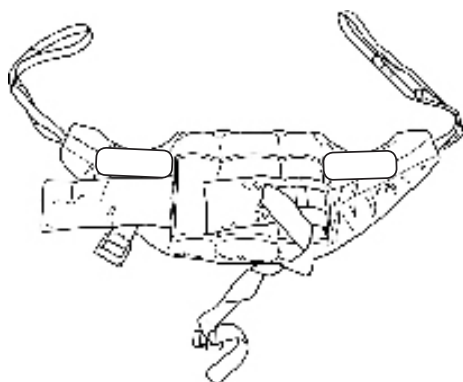

NOTE: DO NOT WASH DISPOSABLE SLINGS



HOYER RECOMMENDS THE USE OF GENUINE HOYER PARTS.

Hoyer slings and lifters are not designed to be interchangeable with other manufacturer's products. Using other manufacturer's products on Hoyer products is potentially unsafe and could result in serious injury to patient and/or caregiver.

For the safety of the patient and carer; before using a sling a full risk assessment must be conducted to ensure that the correct sling choice, method of positioning in the sling and procedure for transfer has been determined for the patient.

Check sling and stitching before each use. Using bleached, torn, cut, frayed or broken slings is unsafe and could result in serious injury or death to the patient.

Destroy and discard worn slings. Do not alter slings.

Never leave a patient unattended.




Additional slings are available to suit individual needs.
You are advised that you should always seek the advice of an authorised dealer before purchasing.

We will be pleased to advise you of your nearest authorised dealer.

**Joerns**

ENGLISH

HOW TO FIT THE DELUXE STANDING SLING

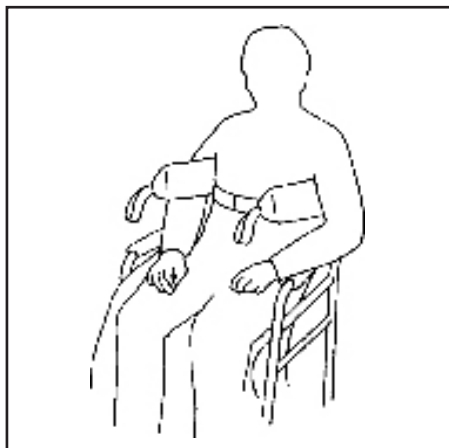

Place sling down behind client's back between base of shoulder blades and bottom of rib cage. Attach strap around the front of the client. This strap need not be tight. It is there to hold the sling in position whilst attaching the sling to the Standaid. Ensure that the client's arms are outside the sling.



Position Standaid so that the client's feet fit in the foot trays and knees rest against the knee pad. Initially attach the longer loops of the sling to the white hooks at the end of the hand grips. The client's hands should be placed on the black hand grips or support handles. (Once the client is used to the lift, the shorter loops may be used to achieve a more upright position). Then push the up button on hand control to raise the client to the standing position.

NOTE: THIS SLING IS NOT A GENERAL-PURPOSE SLING AND WILL NOT BE SUITABLE FOR ALL CLIENTS.

TIPS FOR USE:

CLIENT SHOULD BE ASKED TO LEAN BACK INTO THE SLING FOR THE DURATION OF THE LIFT.

TALLER CLIENTS MAY NEED TO USE SHORTER LOOPS ON THE SLING FROM THE START.


SHOULD THE CLIENT START TO LOSE TRUNK CONTROL YOU WILL FIND THAT THE SLING WILL SLIP UP BENEATH THE ARMPITS MAKING THE LIFT UNSAFE. IN THIS SITUATION THE TRANSPORT SLING CAN BE USED PROVIDING A RISK ASSESSMENT IS COMPLETED.

TECHNICAL SPECIFICATIONS

STANDARD	BS EN ISO 10535
SAFE WORKING LOAD	440LBS / 200KG

SIZING

Each sling is marked with the following colour coded symbols, four of which will be crossed. Your sling is the uncrossed size, i.e. in this example Small (Red).

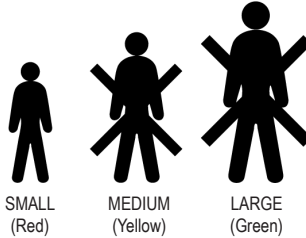

WASHING INSTRUCTIONS

DO NOT WASH

Disposable slings are patient specific and are designed not to be washed. Washing will compromise the integrity of the sling and therefore make it unsafe for use.



SIZE	STANDING DISPOSABLE
SMALL (RED)	NA13525N (Polyester)
MEDIUM (YELLOW)	NA13526N (Polyester)
LARGE (GREEN)	NA13527N (Polyester)

*THESE SLINGS ARE SUITABLE FOR USE WITH HOYER STAND AID LIFTS
DO NOT EXCEED THE RATED CAPACITY OF THE SLING OR THE LIFT*



Joerns Healthcare
 2100 Design Road, Suite 100
 Arlington, TX 76014
 Tel: 800-826-0270
 Fax: 800-457-8827
www.joerns.com