

At Joerns Healthcare, we work with you to help elevate safety standards and enhance care quality. Our safety solutions are focused on preventing falls and reducing injuries to help save your facility unnecessary expenses. In combination with effective use of our comprehensive products, our safety specialists work with your staff to identify and analyze risk, pinpoint and implement suggested solutions, and measure solution success.

We blend our long history of designing and manufacturing healthcare equipment and furnishings, with ergonomic and safety principles, to deliver unique safety solutions that work in your environment. Our safety specialists help you assess your current environment and processes to identify areas of opportunity to optimize your risk management results.

Hoyer Slide Sheets are made of ultra low friction material to assist in repositioning and to help to reduce injuries. Our slide sheets are washable and available in three different sizes to provide three different transfer applications:

- 1) **Small Size**
Small slide sheets are to be used under the resident's feet, along with a large slide sheet, when moving a resident in a prone position.
- 2) **Medium Size**
Medium slide sheets are to be used for positioning the resident's hips and mid-section. Medium slide sheets can also be used for repositioning on smaller and/or shorter residents in the prone position.
- 3) **Large Size**
Large slide sheets are to be used for lateral repositioning of average to tall residents in the prone position.

Why Hoyer Slide Sheets do not have handles:

- Handles promote poor ergonomic posture by forcing the caregiver to grip the slide sheet by handle
- Handles have the potential of harboring infectious bacteria
- High temperature washing and drying can cause handles to distort or shrink, making the handles difficult to use

Hoyer Slide Sheet Part #	Size	Dimensions
NA25525	Small	24" x 18"
NA25526	Medium	36" x 25"
NA25527	Large	60" x 31"

Joerns reserves the right to change specifications without notice.

