


Parts Manual

Hoyer® Elara Ceiling Lifts

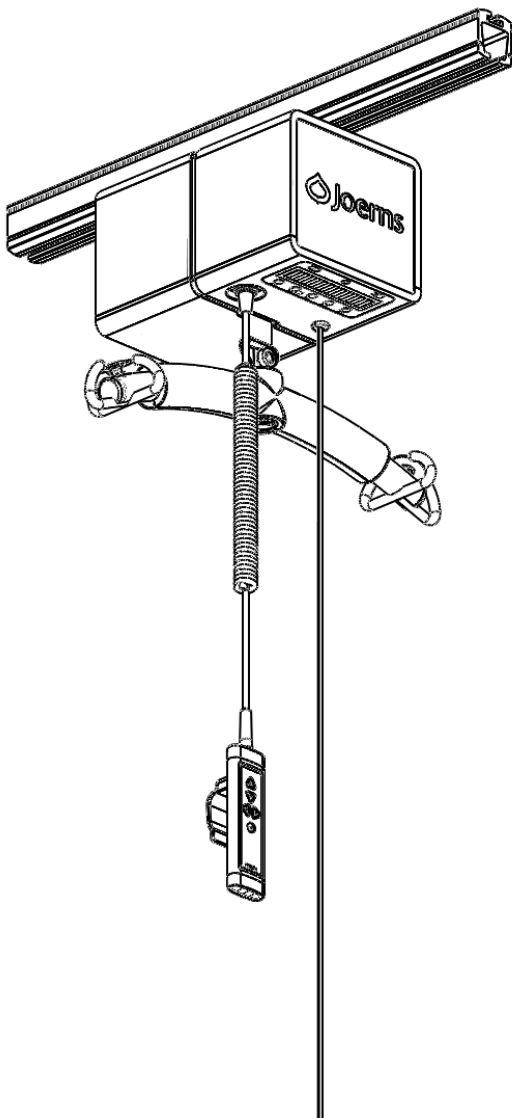

NO	PART NUMBER	DESCRIPTION	QTY
1	39001600CH	Lift Tape Assembly	1
2	39001601CH	Spreader Bar (inc. Central Pivot & Slick Pin)	1
3	39001630CH	Pediatric Spreader Bar Assembly (inc. Central Pivot and Slick Pin)	1
4	39001631CH	Large Spreader Bar Assembly (inc. Central Pivot and Slick Pin)	1
5	39001602CH	Slick Pin Assembly	1
6	39001639CH	US Lower Cover Assembly (inc. LED Strips and Decals)	1
7	39001605CH	Display Screen - 227kg/500lbs	1
8	39001606CH	Display Screen - 284kg/625lbs	1
9	39001607CH	Battery (inc. Fuse, Link Wire and Cable Ties)	2
10	39001608CH	Main PCB (inc. 30 amp Fuse & 2 amp Mini Blade Fuse)	1
11	39001609CH	30 amp Fuse	5
12	39001610CH	20 amp Battery Link Fuse	5
13	39001611CH	2 amp Mini Blade Fuse	5
14	39001612CH	Battery Retaining Strap	1
15	39001613CH	Main Switch Loom (inc. Emergency Stop Switch and Twisted Tape Switch Housing)	1
16	39001641CH	2-way Upper Cover Assembly (inc. x4 1/4 Turn Knob Assemblies & Hoyer Decals)	2
17	39001642CH	4-way Upper Cover Assembly (inc. x4 1/4 Turn Knob Assemblies & Hoyer Decals)	2
18	39001615CH	1/4 Turn Knob Assemblies (inc. 'Clip-Nuts' and Washers)	4
19	39001616CH	Power Train Assembly (inc. Main Switch Loom and SWL Potentiometer)	1
20	39001617CH	2-way Handset	1
	39001618CH	4-way Handset	1
21	39001620CH	Warm Lighting Loom	1
22	31009654	Charge Transformer Kit (US)	1
23	39001625CH	Emergency Stop Cord and Moulding Assembly	1
24	39001626CH	2-way Trolley Complete	1
25	39001627CH	4-way Trolley Complete	1
26	39001628CH	Charge Roller Assembly (Complete) (inc. compression spring and retaining fixing)	1
27	39001629CH	Emergency Lower 4mm Hex Key	1
28	39001632CH	Battery Loom	1
29	39001633CH	Cover Disc	1
30	39001634CH	USB PCB	1
31	39001635CH	USB Ribbon Cable	1
32	39001636CH	Main Ribbon Cable	1
33	39001637CH	Internal Handset Loom	1
34	39001304	Spreader Bar Sling Retaining Washer Kit (inc. Retaining Washer x2, M6 Screw x2, Spring x2, Spacing Post x2)	1
35	11016062	Charger Mains Lead (US)	1
36	39001645CH	Charger Bracket	1
37	0YJH018500	Scale Kit	1
N/S	39001603CH	Wear Washer Kit (inc. 'O' Ring)	1
N/S	39001644CH	Hoyer Elara Decal Kit	1

