


Parts Manual

Hoyer® Journey

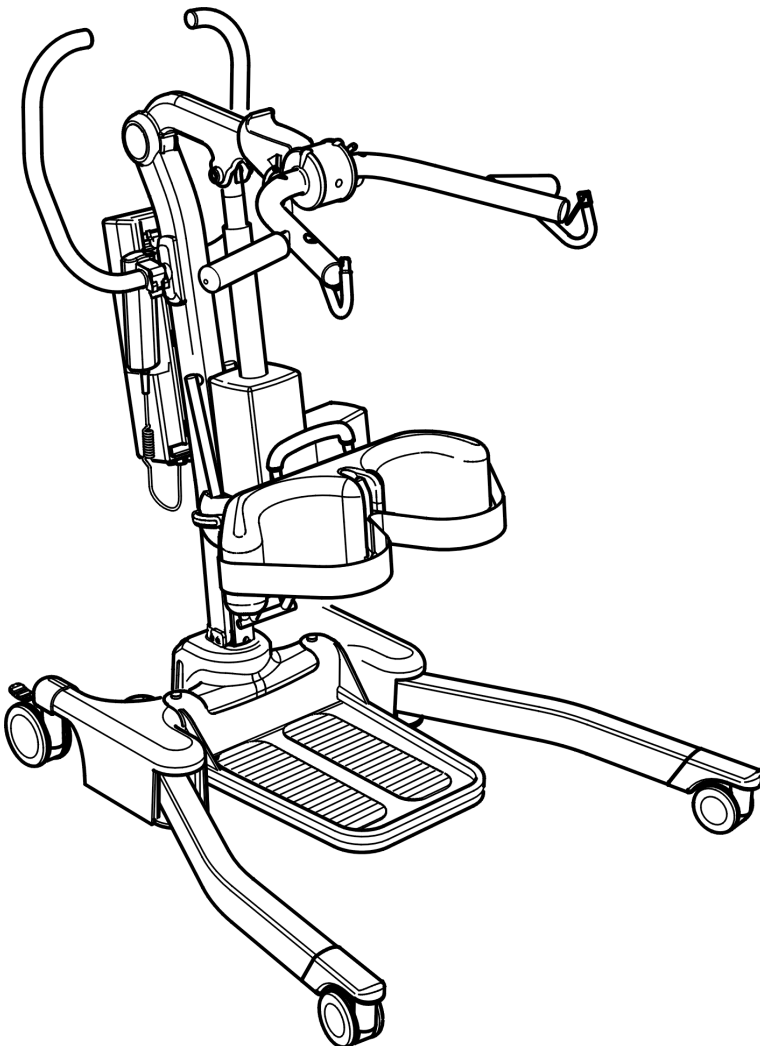

NO	PART NUMBER	DESCRIPTION	QTY
1	0Y0066	100mm rear caster (braked)	2
2	0Y0109	75mm front caster (non-braked)	2
3	0Y0679	Mast assembly (including latch mechanism)	1
4	0Y0663	Cow horn and boom assembly	1
5	0Y0660	Base assembly (inc. base casting, bottom plate, welded post & labels, not internal parts)	1
6	0Y0675	Left leg extrusion (including fixings)	1
7	0Y0676	Right leg extrusion (including fixings)	1
8	39001318	Calibrated control box - Smart™ Monitor	1
	0Y0678	Control box - Standard	1
9	0Y0067	Battery pack	1
10	11014544	User handset (Smart™ Monitor only)	1
	0Y0111	User handset (Standard control box)	1
-	39001369	Actuator reset handset (Smart™ Monitor only) (not shown)	1
11	0Y0053	Charger and stand assembly (without charger lead)	1
12	0Y0114	Charger lead (US)	1
13	0Y0677	Actuator - 210mm stroke (including location pins, sleeves, cover and labels)	1
14	0Y0069	Battery and controller bracket (inc. fixings)	1
15	0Y0669	Foot pedal mechanism complete	1
16	0Y0042	Foot push pad assembly complete	2
17	0Y0045	Front caster leg inserts including fixings (plastic)	2
18	0Y0659	Upper fixings kit	1
19	0Y0661	Base fixings kit (all bolts, screws and fixings)	1
20	0Y0684	Push handle - left hand	1
21	0Y0685	Push handle - right hand	1
22	0Y0683	Push handle covers	2
23	0Y0682	Pivot joint plastic cover (includes inner and outer piece)	2
24	0Y0662	Cow horn adjustment slide cover (left and right part)	1
25	0Y0666	Cow horn indicator cover (left and right part)	1
26	0Y0674	Knee pad rod adjustment clamp	1
27	0Y0671	Knee pad adjustment fixings	1
28	0Y0686	Rear base cast cover	2
29	0Y0665	Cow horn handle grips	2
30	0Y0672	Knee pad adjustment handle	2
31	0Y0500	Leg strap	1
32	0Y0499	Knee pad assembly (including fixing bolts)	1
33	0Y0673	Knee pad bracket (not including knee pad or leg strap)	1
34	0Y0670	Foot tray assembly	1
35	0Y0681	Mast lock pin and handle	1
-	0Y0687	Sling retaining clip (not shown)	10
-	0Y0128	Caster tightening wrench (not shown)	1
-	0Y0667	Decal set (excludes serial number labels) (not shown)	1
-	0Y0070	Handset clip (not shown)	2

