



MANGAR Camel Lifting Cushion

The Camel Lifting Cushion will both sit up and lift a fallen person.

EASY TO FIT

- Lifting cushion is inflated using an Airflo Compressor with a simple hand control
- Simple to use, requiring very little training
- Suitable for indoor and outdoor use
- Can be used independently or assisted

SIMPLE TO MAINTAIN

- Clean using a proprietary non-abrasive mild liquid cleaner.
- Easy to deflate, fold away and store in the carry bag provided
- The material is designed to exceptionally high standards with extraordinary durability, strength & performance in the toughest of environments

SAFE TO USE


- Lifts the frailest person up to 700 lbs from the floor
- Stable and supportive
- Ideal for caregivers of individuals with neurological disorders who fall regularly

IMPROVES WELL-BEING

- Provides a safe, dignified lift for a fallen person
- Reduces the risk of musculoskeletal injury for the caregiver
- Individuals with a cognitive impairment are less distressed when lifted with a Camel
- The backrest gives additional support and is particularly suitable for bariatric patients

TECHNICAL SPECIFICATIONS

Max user weight	700 lbs
Max height	22"
Min height	Flat
Seat depth	17.5"
Total length	44"
Width	28.5"
Total product weight	14 lbs
Back/headrest length	32"
Airflo Compressor	24
Lifting performance per charge	Up to 4 lifts

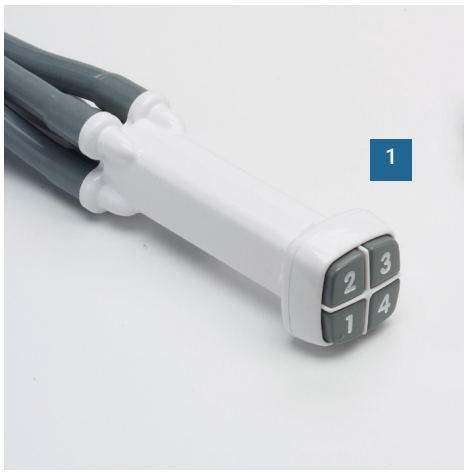

Features

1. **Easy to setup** - connect the four hoses from the hand control to the corresponding numbers and coloured connectors on the Camel.
2. **The head should lie on the Camel badge** and the body should lie centrally.
3. **The backrest** may be partially inflated before the lower sections.
4. **Made from very strong polyurethane coated nylon material.**
5. **Each layer inflates individually** ideal for single-handed care.



Product code: MPCA060400

Accessories

- Slidesheet
- Camel carry bag



Joerns.com

General 800.826.0270

Canada 866.546.1151

6240085 RevA • 000576

US Partner with

 **Joerns**[®]
HEALTHCARE

WIN  **CARE**

