



# User-Service Manual

## Joerns® Support Surface

### CairTurn RT

To avoid injury, read user's manual before using.



## Important Precautions

**Important Notice:** The equipment must be installed and operated in the manner for which it was intended. Facility staff/user is responsible for reading and understanding the product user manual and contacting Joerns if anything in this manual is unclear. Joerns will not be held responsible for any injuries resulting from failure to comply with the instructions and precautions in this manual.

**Warning:** Joerns' specialty support surfaces are designed as mattress replacement systems. The risk of entrapment may occur when the equipment is placed on bed frames that leave gaps of even a few inches between the mattress and the headboard, footboard, and bed or side rails. The equipment is NOT to be used when such gaps are present.

Facility staff/user is responsible for ensuring that all mattresses properly fit the bed frames. Joerns is not responsible for the placement of its equipment on bed frames that leave gaps between the mattress and the headboard, footboard or bed or side rails which present a risk of harm to patients.

**Warning:** The safe use of the equipment is maximized when used in conjunction with bed rails; there may be an increased risk of falls when such bed rails are not present. Serious injury or death can result from the use (potential entrapment) or non-use (potential patient falls) of side rails or other restraints. Local policies regarding the use of side rails should be taken into account. Whether and how to use side rails is a decision that should be based on each patient's individual needs and should be made by the physician, caregivers and responsible parties.

When using the mattress system, always ensure that the patient is positioned properly within the confines of the bed. Do not let any extremities protrude over the side or between the bed rails when the mattress is being used.

**Danger Explosion Hazard:** Do not use in the presence of flammable anesthetics. Do not use in the presence of smoking materials or open flame. Air flowing through the air mattress will support combustion.

**Danger:** To reduce the risk of shock, adhere to the following instructions. Failure to do so could result in personal injury or equipment damage.

- Immediately after using the CairTurn RT, unplug it from its power source.
- Do not place or store the product where it can fall or be pulled into a tub or sink.
- Do not place or drop the product into water or other liquid.
- Do not remove the back of the control unit. Refer servicing to Joerns.

**Warning:** To reduce the risk of burns, shock, fire, or personal injury, adhere to the following instructions. Failure to do so could result in personal injury or equipment damage.

1. Use this product only for its intended purpose as described in this manual. Only use attachments and/or accessories that are recommended by the manufacturer.
2. If this product has a damaged power cord or plug, is not working properly, has been dropped or damaged, or has been dropped into water, do not operate it. For examination and repair, return the product to Joerns.
3. Keep the control unit and power cord away from heated surfaces, e.g. space heaters.
4. Never block the air openings of the product. Do not place the control unit on a surface, such as a bed or couch, where the air opening and/or filter compartment, located on the back of the control unit, may be blocked. Keep the air openings free of lint and hair.
5. Never drop or insert any object into any opening or hose.
6. Do not spill food or liquids onto the control unit. If a spillage does occur, turn off the unit, disconnect it from its power supply and allow at least 24 hours for drying.
7. Do not use the product outdoors, or where aerosol-spray products are used.
8. Plug this product only into a properly grounded outlet. Refer to "Grounding Instructions".

9. Ensure nothing is placed on the power cord and ensure it is not located where it can be stepped on or tripped over.
10. Do not attempt to service the control unit. Please call Joerns for any service requests.
11. The therapy pad (top cover) of this product is not air permeable and may present a suffocation risk. It is the responsibility of the caregiver to ensure that the patient can use this product safely.

## Save These Instructions For Future Reference

### Bed System Entrapment Information

Although common in the practice of long-term care, bedside rails, in recent years, have also been a subject of regulatory review and evolution in design and use.

That focus includes not only the challenge of achieving an appropriate balance between patient security and unnecessary restraint, but also the additional safety issue of entrapment.

The U.S. Food and Drug Administration (FDA), working with Joerns Healthcare and other industry representatives, has addressed the potential danger of entrapment with new safety guidelines for medical beds. These guidelines recommend dimensional limits for critical gaps and spaces between bed system components.

Entrapment zones involve the relationship of bed components often directly assembled by the healthcare facility rather than the manufacturer. Therefore, compliance is the responsibility of the facility.

As the leading manufacturer of long-term care beds and a frontrunner in addressing this critical issue, Joerns Healthcare can offer you the expertise, assistance and products to bring your facility into compliance.

### Joerns Compliance Solutions

Matching the right bed components in order to meet regulatory guidelines can be complex.

That's why Joerns offers a wide array of compliance options. We assist customers in selecting compliant accessories recommended for their specific bed model.

### Creating a Safer Care Environment

While the guidelines apply to all healthcare settings (hospitals, nursing homes and home care), long-term care facilities have particular exposure since serious entrapment events typically involve frail, elderly or dementia residents.

### For More Information

To learn more about compliance options with Joerns products, visit our website at [www.joerns.com](http://www.joerns.com), or contact our Customer Care representatives at 800.826.0270 and ask for free informational publications.

To learn more about entrapment zones, assessment methods, and guidelines concerning entrapment, contact Joerns Healthcare at 800.826.0270 or consult the FDA website: [www.fda.gov/cdrh/beds](http://www.fda.gov/cdrh/beds).

|   |             |
|---|-------------|
| <b>Important Precautions</b> .....                      | <b>2</b>    |
| <b>Introduction</b> .....                               | <b>5</b>    |
| Moisture Control  |             |
| Shear and Friction Reduction                            |             |
| <b>Indications for Use</b> .....                        | <b>5</b>    |
| Spinal Cord Injury                                      |             |
| Pressure Relief   |             |
| Pain Management   |             |
| <b>Features</b> .....                                   | <b>6</b>    |
| Therapy Control Unit Features                           |             |
| Therapy Mattress System Features                        |             |
| Therapy Pads  |             |
| <b>Grounding Instructions</b> .....                     | <b>7</b>    |
| <b>Setup</b> .....                                      | <b>7</b>    |
| <b>Operation</b> .....                                  | <b>8</b>    |
| Patient Comfort Controls and Monitoring                 |             |
| <b>Additional Features</b> .....                        | <b>9</b>    |
| CPR   |             |
| Transport   |             |
| Power Failure   |             |
| Optional Accessories                                    |             |
| <b>Troubleshooting</b> .....                            | <b>9</b>    |
| Therapy Surface Not Inflating                           |             |
| Unable to Change Therapy Mode or Adjust Comfort Control |             |
| <b>Nursing Procedures</b> .....                         | <b>9-10</b> |
| Recommended Linen                                       |             |
| Changing the Therapy Pad                                |             |
| Resident Positioning and Comfort                        |             |
| Incontinence  |             |
| <b>Safety information</b> .....                         | <b>10</b>   |
| Patient Migration                                       |             |
| Traction  |             |
| Skin Care   |             |
| Bed Height  |             |
| <b>Cleaning</b> .....                                   | <b>11</b>   |
| Control Unit  |             |
| General Cleaning  |             |
| Disinfecting  |             |
| Therapy Pad   |             |
| Steam Cleaning  |             |
| <b>Maintenance</b> .....                                | <b>12</b>   |
| <b>Storage and Care</b> .....                           | <b>12</b>   |
| Control Unit  |             |
| Support Surface   |             |
| <b>System Specifications</b> .....                      | <b>12</b>   |
| Weight  |             |
| Dimensions  |             |
| Electrical Specifications                               |             |
| Environmental Conditions                                |             |
| Agency Approvals  |             |
| <b>Call for Assistance</b> .....                        | <b>12</b>   |
| <b>Warranty</b> .....                                   | <b>14</b>   |



## Introduction

The CairTurn RT, provided by Joerns, is a unique bilateral rotation therapy mattress replacement system. This system is designed to meet pressure relief and rotational therapy needs of patients up to 350 lbs. in weight with pressure ulcers, special respiratory needs, and those at risk of skin breakdown.

**⚠ Warning:** The risk of entrapment can arise when equipment is placed on bed frames that leave gaps of even a few inches between the mattress and the headboard, footboard, and bed or side rails. The equipment is NOT to be used when such gaps are present. See “Important Precautions” section of this manual.

The CairTurn RT mattress replacement system is suitable for both the treatment of existing pressure ulcers stage I through stage IV as well as those who have been assessed at risk from the complications of immobility. The CairTurn RT is quiet, comfortable and simple enough for single caregiver installation. The user-friendly controls allow for easy adjustment of patient comfort.

Additionally, low friction and low shear materials, together with average interface pressures well below capillary closure levels, means that the CairTurn RT meets the comfort and clinical requirements of your patients up to 350 lbs.

We have ensured that the CairTurn RT addresses the four key areas in the treatment of compromised skin: pressure relief, moisture control, and reduction in both friction and shearing forces.

### Moisture Control

Patients are at risk for skin maceration if excess moisture is permitted to accumulate beneath the patient. This may be due to perspiration, incontinence or wound drainage.

On the CairTurn RT, moisture is controlled via the specially treated breathable, fluid-proof, urethane coated nylon therapy pad. The moisture vapor permeable fabric of the therapy pad allows a sufficient amount of air to circulate beneath the pad and wicks away excess moisture.

## Shear and Friction Reduction

Shearing occurs when the skin is stationary in relation to the support surface, while the underlying tissues and vessels are stretched and damaged. When a patient’s skin rubs against another surface, the result is friction. The top surface of the CairTurn RT therapy pad is constructed from a very smooth nylon fabric with low friction and low shear properties to protect the patient’s skin from these damaging forces.

## Indications for Use

**Note:** The selection of a pressure-relieving surface needs to be based on each individual patient’s clinical condition, diagnosis and/or co-morbidities. The choice and use of a support surface is one factor in a holistic program of wound care and treatment.

### Spinal Cord Injury

The CairTurn RT is not recommended for use by patients with unstable spinal fractures. Advice should be obtained from the appropriate physician before using the CairTurn RT system for these patients.

### Pressure Relief

|                 |                        |
|-----------------|------------------------|
| Pressure Ulcers | Respiratory Conditions |
| Rehabilitation  | Neurology              |
| Dermatology     | Burns                  |
| Amputations     |                        |

### Pain Management

|          |           |
|----------|-----------|
| AIDS     | Arthritis |
| Oncology |           |

## Joerns® Support Surface CairTurn RT

The CairTurn RT provides distribution of weight over a wide surface area, which relieves pressure against bony prominences and provides a soft, gentle therapy surface to lie on. For patients experiencing severe pain and discomfort due to pressure and/or positioning limitations, consider the CairTurn RT as an adjunct to pain management interventions.

**Note:** Pressure relief and pain management are conditions and diagnoses for which the CairTurn RT may be indicated. Occasionally, there are orthopedic and neurological patients that require body positioning to be maintained in specific alignment. The use of the CairTurn RT for these patients should be considered on an individual basis and discussed with the attending physician.

### Features

The CairTurn RT is comprised of two components:

- Therapy control unit
- Therapy mattress system

### Therapy Control Unit Features

- Eight comfort control settings to maximize patient compliance and promote healing.
- Three modes of operation – *Autofirm*, *Therapy* (static) and *Turning Therapy*
- *Autofirm* mode provides maximum air inflation designed to assist both patients and caregivers during patient transfer and treatment.
- Six *Turn Cycle* times: 20, 30, 45, 60, 90 and 120 minutes
- Preset *Turn Angle* for optimal rotation therapy
- Compact lightweight control unit is quiet, robust and powerful, with an integrated carrying handle for portability.
- Crisp, easy to read graphics for intuitive set up and therapy control.
- Automatic panel lock out to avoid unwanted or accidental adjustments.
- Closed loop pressure sensor control system eliminates concerns of changes in mattress interface pressure due to ambient temperature and pressure changes.
- Specially designed enclosure for low noise and durability.
- Integrated swing out hanging brackets for fixing to most bed types.

- Quick disconnect hose feature allows for rapid attach and CPR deflate at the control unit.

### Therapy Mattress System Features

- Twenty individual therapy cells help to evenly distribute the patient's weight and maximize pressure relief.
- Modular cell design mattress for ease of cleaning, re-assembly and cost effective service.
- Lower two-inch therapy support cell provides additional therapeutic support and remains inflated for up to 12 hours in the event of a power failure.
- Lower mattress therapy enclosure provides for stable ingress and egress and reduces entrapment concerns
- Eight-inch deep therapy cells are constructed of highly durable, polyurethane coated nylon to provide adequate support and prevent bottoming out for most patients within weight limit.
- Side bolsters support patient during rotation
- Durable base tub is constructed from 100% heavy weight 1680 denier nylon with a 1.5 oz. urethane coating and incorporates bed attachment loops for stability.
- Maximum weight capacity of 350 lbs.
- Quick CPR deflation
- Anti-kink, easy clean air supply hose set

### Therapy Pads

Many healthcare facilities are facing the challenge of infection control. Joerns' quilted therapy covers are treated with an antimicrobial agent to protect the therapy pad from the growth of mold, mildew and bacteria.

#### Key features and benefits:

- Treated with a highly effective bacteriostat agent to inhibit the growth of bacteria and fungus.
- Constructed from a very smooth nylon fabric with low friction and low shear properties to protect the patient's skin from damaging friction/shearing forces.
- Breathable, moisture vapor permeable fabric allows air to circulate beneath the pad and wicks away excess moisture. This keeps your patient dry and helps to prevent skin maceration.
- Two-way stretch therapy pad is designed for optimal comfort, moisture vapor transfer, stain resistance and ease of laundering.

## Grounding Instructions

**⚠ Warning:** Use a properly grounded, three-prong, 120V AC outlet for this product. Failure to use a grounded outlet could result in personal injury or damage to equipment or house wiring, including risk of fire. A qualified electrician should be contacted to correct the wiring and ensure a properly grounded outlet.

Before installing this product, have the electrical system checked to make sure the electrical circuits and the electrical service are properly grounded.

Having a three-prong outlet does not necessarily mean it is grounded. Sometimes two-prong outlets are replaced with a three-prong type even though there is no ground wire.

There is always a chance of a loose connection or poor installation of a ground wire that causes the loss of proper ground at the outlet. Inadequate grounding at electrical outlets can occur even if there is a ground wire. Wires can become loose over time at the connection to the outlet.

**Note:** To install new wires on a circuit requires a qualified electrician.

### How to Determine if Your Outlet has the Proper Grounding

Most hardware stores sell circuit testers (Figure 1) that can be used to test an outlet for proper grounding. The tester plugs into an outlet and by observing the indicator lights you can determine if the outlet is properly grounded. For higher level of assurance, an electrician should be requested to thoroughly test the electrical system with more reliable equipment.



Figure 1

If repair or replacement of the cord or plug is necessary, please contact Joerns at 800.966.6662 for assistance.

## Setup

**⚠ Warning:** For important precautions please see page two.

**Caution:** Do not place the control unit on the floor. Position the power cord to keep personnel from tripping over it.

**Note:** The CairTurn RT system must be installed on bed frames that are equipped with side rails. Please raise side rails on the bed and lock them in position after the patient is on the mattress. Never leave patient unattended on mattress system with bed side rails in the down position.

When the product is not in use, properly store the power cord. Failure to do so could result in personal injury.

- Remove the existing mattress from the bed.
- Unroll the mattress with the hose connection at the foot end of the bed and the therapy cells facing up. Secure the straps on the mattress securely to the movable part of the bed frame.
- If the therapy pad is not already on the mattress, place it on the mattress. Secure the therapy pad to the enclosure with the provided zipper.
- Hang the control unit on the foot of the bed facing away from the bed. Attach the hose connectors to the control unit.
- Plug in the control unit and the yellow *Standby* light will illuminate. Press the *Power* button. The control unit will start and the green light will illuminate. Keep the control unit ON while the patient is on the mattress.
- Allow thirty to forty minutes for full inflation. Place the patient on the mattress.
- Position the resident's head in the position that the resident will be in for the largest portion of the day. If the resident is lying flat, please use three (3) fingers for the hand check. If the resident will be sitting up for the majority of the day, please use two (2) fingers.
- Perform a hand check by placing fingers locally under the resident's buttocks between two cushions. The resident should not bottom out. If they do, increase the therapy control by one LED (light), until they no longer bottom out.

## Operation

**Warning:** For important precautions please see page two.

**Caution:** The patient's head should be positioned in the center of the top section of the mattress. When using the mattress system always ensure that the patient is positioned properly within the confines of the bed. Do not let any extremities protrude over the side or between the bed rails when the mattress is being used.

## Patient Comfort Controls and Monitoring (Figure 2)

### Power Switch

The power switch is used to turn the power on and off.

### Standby

When the *Standby* light is on, it indicates that there has been a power interruption and the therapy control unit is ready to be turned on. Press the *Power* button and set the preferred mode of therapy and comfort level.

## Modes

### Autofirm

*Autofirm* mode provides maximum air inflation designed to assist both patients and caregivers during patient transfer and treatment. The unit will automatically return to the *Therapy* mode within approximately 12 minutes.

### Therapy Modes: Therapy and Turning

- *Therapy*: The unit starts in the *Therapy* mode, which is the standard pressure relief mode.
- *Turning*: The *Turn* mode provides a continuous rotation cycle based on selected times and a preset angle.

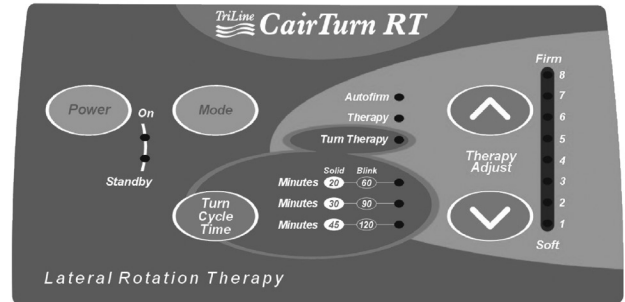


Figure 2

### Turn Cycle Time

The CairTurn RT provides a dynamic turning support surface for the patient. The patient will turn laterally. The cycle starts from center (flat) and turns fully to the left, the right, and then back to the center in the chosen cycle time (20, 30, 45, 60, 90 and 120 minutes). This lateral rotation sequence will continuously repeat until the mode or *Turn Cycle* time is changed.

### Therapy Adjust

The *Therapy Adjust* function is located on the right side of the control panel. The CairTurn RT can be customized to meet individual patient needs within a therapeutic window. Use the up and down keys to simultaneously increase or decrease pressure in the entire mattress. This function will not work in *Autofirm* mode.

### Lockout

This feature is to prevent any unauthorized changes to the patient settings. To unlock and make adjustments to the settings, press both up and down comfort arrows at the same time to disengage the *Lockout* function. The *Lockout* function will return in approximately five minutes.

**Note:** The unit is designed to lock out all of the adjustment controls after the patient has been positioned correctly. In approximately five minutes after the last button has been pushed, push the power on light begins to flash indicating *Lockout* is enabled.



## Additional Features

**⚠ Warning: A possible fire hazard exists.** This product is suitable for use with oxygen administering equipment of the nasal, mask, or half bed-length, tent-type only. To prevent personal injury or equipment damage, ensure that the oxygen tent does not extend below the mattress.

### CPR

The hose connection at the control unit is marked *CPR*. Disconnect the hose from the control unit. Deflation times will vary based on patient weight and profile. To resume therapy, reattach the hose to the control unit.

### Transport

To transport the patient in bed, turn the control unit off. Unplug the power cord from the outlet. Do not disconnect the hose connection at the control unit. The lower support cell will prevent the patient from bottoming out for up to 12 hours.

### Power Failure

In the event of a power failure the mattress will stay inflated for up to 12 hours while connected to the control unit. After power has been restored, the control unit's Standby light will be lit. Press the power button and reset the preferred mode and comfort level as described in the "Operation" section.

### Optional Accessories

- Additional therapy pads available for purchase

## Troubleshooting

### Therapy Surface is not Inflating

1. Ensure the hose connection from the therapy mattress system (mattress) to the control unit is securely connected.
2. Ensure that the control unit is plugged into an AC outlet.

3. Ensure that the power is not on *Standby*. If on *Standby*, press the *Power* button.
4. Ensure that all air cells are connected to the manifold.

### Unable to Change Therapy Mode or Adjust Comfort Control

Make sure the *Lockout* function is disabled. To disable, press the up and down comfort adjust arrows simultaneously.

## Nursing Procedures

### Recommended Linen:

Special linens are not necessary for the CairTurn RT. There is no need for a bottom sheet as the therapy pad should be covering the therapy cells at all times. The patient should never be lying directly on the therapy cells. Depending upon the patient's specific needs, the following linens may be utilized:

- Draw or slide sheet to aid in positioning and to further minimize friction and shearing
- Incontinence barrier pad for patients incontinent of urine and/or stool, and patients with heavily draining wounds
- Add top sheet, blanket and/or bedspread as needed for patient comfort
- Keep the amount of padding between the patient and bed to a minimum for optimum performance

### Changing the Therapy Pad

- Place the therapy pad over the therapy cells, fitting the corner of the cushions into the corner of the therapy pad. (Similar to a fitted sheet)
- Zip the therapy pad along each side of the mattress tub.

## Patient Positioning and Comfort

### General Repositioning

Patients should be turned and repositioned per individual turning schedule or per facility policy. It may be helpful to activate the *Autofirm* mode to achieve a firm surface for repositioning purposes. The unit will automatically return to the mode it was in prior to *Autofirm* within approximately 12 minutes or you can manually return to therapy mode once patient has been repositioned.

Unless counter indicated, it is desirable to keep the head of the bed in the low position to provide optimal pressure relief and minimize the risk of shearing injuries.

### Elevating Patient into Sitting Position

The special properties of the CairTurn RT therapy pad reduce the opportunity for shear and friction that may occur when raising the head of other beds. As with any surface, sliding can be expected, therefore patients should be repositioned after elevation. The knee gatch or foot of the bed may be elevated first, to help prevent the patient from sliding when the head of the bed is elevated.

### Incontinence

Moisture against the skin surface leads to maceration, or softening of the tissues. To prevent maceration, it is recommended that an incontinence barrier pad be used to absorb the excess moisture.

In the event of incontinence or excess drainage on the therapy pad, wipe off the excess fluid from the bed surface.

## Safety Information

### Patient Migration

Specialty bed products are designed to reduce/relieve pressure and the shearing/friction forces on the patient's skin. The risk of gradual movement and/or sinking into hazardous positions of entrapment and/or inadvertent bed exit may be increased due to the nature of these products.

### Traction

With any traction or unstable fractures, maintain physician-directed angle of articulation and guard against risks of patient migration or inadvertent deflation of patient surface.

### Skin Care

Monitor skin conditions regularly, particularly in areas where incontinence and drainage occur or collect, and consider adjunct or alternative therapies for high acuity patients. Early intervention may be essential to preventing serious skin breakdown.

### Bed Height

To minimize the risks of falls or injury, the patient surface should always be in the lowest practical position when the patient is unattended. Make sure areas under and around the frame are clear of objects, persons and parts of body before adjusting height.

## Cleaning

**⚠ Warning:** Unplug the control unit from its power source. Failure to do so could result in personal injury or equipment damage.

**⚠ Warning:** Do not expose the unit to excessive moisture that would allow for liquid pooling. Personal injury or equipment damage could occur.

**Caution:** Do not use harsh cleansers/detergents, such as scouring pads and heavy-duty grease removers, or solvents, such as acetone. Equipment damage could occur.

### Control Unit

Wipe off dust. If necessary, clean the housing exterior with a disinfectant solution or a mild detergent and a damp cloth. Then wipe dry.

### General Cleaning

If there is no visible soilage with possible body fluids, it is recommended that the mattress system be cleaned with a mild detergent and warm water. If disinfection is desired, a combination cleanser/disinfectant may be used as explained in "Disinfecting" area.

- Patient care equipment that does not come into contact with mucous membranes or non-contact skin, requires low-level disinfection. Wiping surfaces with a properly prepared detergent or disinfectant carries out low level disinfecting.
- Processing of dirty patient care equipment should take place in a designated area away from clean or sterile supplies and food preparation areas.
- Detergent/disinfectants should not be mixed with other germicides or detergents. Using the proper dilution insures the most effective killing power of the disinfectant.
- Wash hands often and well, including after removal of gloves.
- Patient care equipment that is used in isolation areas should be disinfected in accordance with all internal policies and procedures regarding such equipment.

## Disinfecting

When there is visible soilage, and between patients, it is recommended that the unit and mattress be disinfected with a tuberculocidal disinfectant.

Disinfectant should be registered with the Environmental Protection Agency (EPA).

- Use rubber gloves and eye protection.
- Prepare detergent/disinfectant (registered by EPA as hospital disinfectant) solution according to instructions on label for correct use-dilution.
- With support surface deflated, thoroughly wipe down entire mattress, as air cells will lie flat. Be sure to reach all areas underneath and in-between air cells. Allow to air dry.
- If dust or other soiling has accumulated along air hoses, remove using swabs moistened with detergent/disinfectant as necessary. Allow all components to air dry. Wrap mattress in plastic and return to storage area.
- Thoroughly wipe down outside of control unit and allow to air dry. Cover with plastic and return to storage area
- Remove gloves and dispose; wash hands.

### Therapy Pad

The therapy pad can be wiped down with a disinfectant solution or a mild detergent with a damp cloth. If heavily soiled, the therapy pad can be laundered in a washer and dryer with warm water (no more than 120° Fahrenheit). A non-bleach detergent should be used sparingly. Wipe dry or allow to air dry.

### Steam Cleaning

Do not use any steam cleaning device on the unit. Excessive moisture can damage mechanisms in this unit.

## Maintenance

**⚠Warning:** Only facility-authorized personnel trained by Joerns should perform preventative maintenance. Preventative maintenance performed by unauthorized personnel could result in personal injury or equipment damage.

Any maintenance done without Joerns' authorization will invalidate any warranties on this product.

## Storage and Care

**Note:** Clean the CairTurn RT as described in the previous section prior to storage.

### Control Unit

The power cord may be wrapped around the unit for convenience. Wrap the unit in a plastic bag for dust resistance then store the unit in an area appropriate for an electronic medical device.

### Support Surface

Gently roll up the support surface, expelling any residual air, for temporary storage. The mattress should be wrapped in plastic and/or a clean bag for storage.

## System Specifications

### Weight

Control unit: .....12 lbs. (5.4 Kg)  
Mattress:.....22 lbs. (10 Kg)  
Maximum weight capacity: ..350 lbs. (159 Kg)\*

### Dimensions

#### Control unit:

12" (30 cm) W x 9" (23 cm) H x 6.5" (16.5 cm) D

#### Mattress:

36" (91 cm) W x 80" (203 cm) H x 10" (25 cm) D

### Electrical Specifications

#### USA

115V AC, 60 Hz, 0.5A

### Environmental Conditions

#### Operating Conditions

Ambient Temperature: +10°C to +40°C  
Relative Humidity: 30% to 75% Non-Condensing

#### Storage and Shipping Conditions

Ambient Temperature: -10°C to +40°C  
Relative Humidity: 10% to 100%

### Agency Approvals

- UL Classified Medical Equipment UL 60601-1  
Can/CSA C22.2 No. 601.1

UL Classification refers to the power unit only, not the complete mattress replacement system.

### Call for Assistance

If you have any questions or require service on a Joerns product, please call Joerns at 800.966.6662

\* Mattress weight capacity only; total weight must not exceed bed frame manufacturers' specified load capacity.



Notes:



## Joerns Healthcare Warranty Program for Upholstered Furniture

Joerns warrants the CairTurn RT mattress to be sold free from defects in workmanship and materials, under normal and proper use, for a period of two (2) years on the mattress, and one (1) year on the cover and electromechanical mattress components (compressors, valves, printed circuit boards, hoses, and couplers). Damages arising from improper use will not be covered by this warranty.

Improper use is defined as, but not limited to, those caused by:

- Burns
- Use of improper chemical agents
- Needle punctures, cuts, or abrasions
- Excessive loads
- Staining
- Negligent or excessive usage
- Improper maintenance, handling and/or cleaning
- Failure to use in the manner indicated in the CairTurn RT user manual

Any modification, repair or alteration done to the CairTurn RT that was not authorized in writing by Joerns will void this warranty.

Damage caused by use in unsuitable environmental conditions, abuse or failure to maintain the product in accordance with user and service instructions is not covered.

This warranty is extended to the original purchaser of the equipment.

## Parts

Joerns' CairTurn RT contains various parts that wear from normal use. Joerns' obligation under this warranty is limited to supplying replacement parts, servicing or replacing, at its option, any product which is found by Joerns to be defective. When requested by Joerns, parts must be returned for inspection at the customer's expense. Credit will be issued only after inspection.

## Service

Most service requests can be handled by the facility Maintenance Department with assistance from the Joerns Product Service Department.

Most parts requested can be shipped next day air at the customer's expense.

Should a technician be required, one will be provided by Joerns, at our discretion. Only the Joerns Product Service Department can dispatch authorized technicians.

**Manufactured by:**  
Joerns Healthcare, LLC  
2100 Design Rd.  
Arlington, TX 76014

**Post Acute, Acute, HomeCare**  
2430 Whitehall Park Dr. Ste 100  
Charlotte, NC 28273  
(P) 800.826.0270  
(F) 800.457.8827

**VA/Government**  
19748 Dearborn Street  
Chatsworth, CA 91311  
(P) 800.966.6662  
(F) 800.232.9796

**Canadian Office**  
1000 Clarke Rd. Ste 6  
London, ON Canada N5V 3A9  
(P) 866.546.1151  
(F) 519.451.8662

**United Kingdom and  
Other Countries**  
+44 (0)844 811 1156  
+44 (0)844 811 1157

**Netherlands**  
+31 (0)30 6363700  
+31 (0)30 6363799